# TS 130

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## Table of contents

Preface .....	8
Declaration of conformity .....	8
Service life .....	8
Disposal .....	8
Identification plate .....	8
Reuse .....	8
Warranty .....	8
Parts of the product .....	9
Intended purpose .....	9
Indication .....	9
Scope of delivery .....	10
Safety instructions .....	10
Inserting the feet .....	10
Replacing the seat .....	10
Information on use .....	10
Cleaning   Maintenance .....	10
Technical data .....	10

## Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

## Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

## Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

## Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

## Identification plate

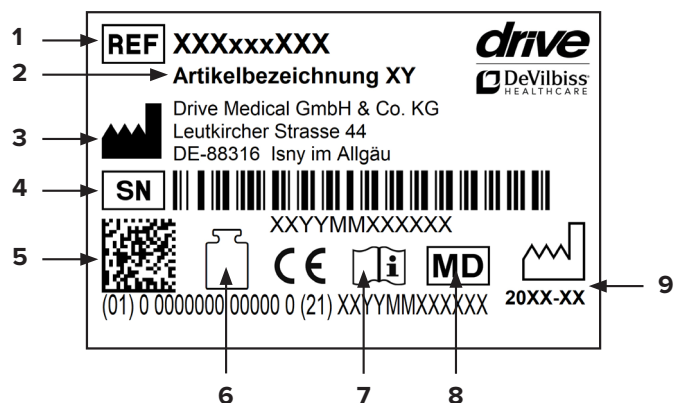1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

## Reuse

This product can be reused. Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at [www.drivedevilbiss.de](http://www.drivedevilbiss.de)

## Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

### EXCLUSION OF WARRANTY

Parts subject to wear include (among others):  
Rubber feet

## Parts of the product

- |                            |                               |
|----------------------------|-------------------------------|
| 1. Back rest               | 5. Standard bucket with cover |
| 2. Armrest                 | 6. Adjustable feet            |
| 3. Seat section, removable | 7. Retaining clips            |
| 4. Seat                    |                               |

### Intended purpose

Toilet aids enable or increase independence when using the toilet and the related personal hygiene, particularly because they compensate for the impaired or missing functions of the motor system or muscular support structure. Commodes are required when using or getting to the toilet is not possible due to the disability. Commodes can also be used as room chairs. The commode can also be positioned over the toilet.

### Indication

#### INDICATION:

Significant to full impairment of mobility/walking and transfer in the case of structural and/or functional damage to the lower extremities (including amputation, after-effects of injuries, musculoskeletal/neuromusculoskeletal movement disorders)

- For use of the integrated toilet unit or to position over a toilet.

## Scope of delivery

1 x	commode, preassembled
1 x	bucket with cover
1 x	instructions for use

## Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- Avoid wet conditions – risk of slipping!
- Prevent unauthorized use, for example, by children.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the type label.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.


## Inserting the feet

The commode is preassembled. Adjust the feet such that the commode is at your preferred height (in six 20-cm increments from 520-640 mm) and secure the feet by inserting the retaining clips as shown (photo 1-2).

## Replacing the seat



The toilet seat is only clipped on and can be removed upwards with corresponding force (photo 3-4).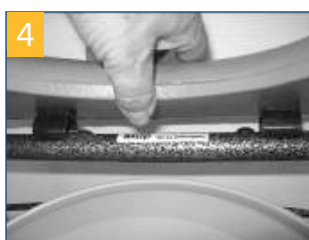

## Information on use

The commode can be used like a regular chair. For use as a commode, remove the padded seat section, then fold up the toilet seat and take the cover off the bucket. Put the toilet seat back down.

After use, remove the toilet bucket by the carrying handles for cleaning and pull it up and out. It is also possible to remove it from either side (photo 5-6).

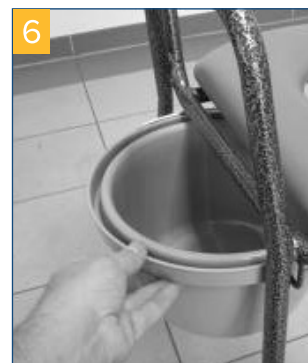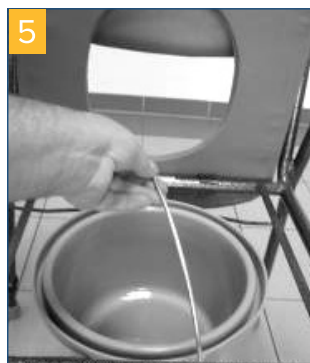## Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary!
- The product must not be continuously exposed to temperatures over + 60°C (e.g. use in a sauna, etc.)
- In the event of defects, contact your specialist dealer from whom you purchased this product.

## Technical data

Product	TS 130
Article number	530100100
HMV (medical aids register) no.	33.40.04.0016
Overall length	57 cm
Overall width	58 / 62 cm
Overall height	83,5 - 98,5 cm
Seat width	43 cm
Seat depth	46 cm
Weight	8,5 kg
Max. load	130 kg
Frame material	Welded steel tubing, hammered paint
Seat / armrest cover material	Wood with foam and flame-resistant PVC cover

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Drive Medical GmbH & Co. KG