tGo

(DE)	Bedienungsanleitung – tGo Gehtrainer	6
	→ Download aktuelle Bedienungsanleitung: www.thomashilfen.de	
(GB)	User manual – tGo gait trainer	14
	→ Download the current user manual: www.thomashilfen.com	
(ES)	Manual del usuario – tGo andador	22
	→ Descarga del manual actual: www.thomashilfen.com	
(PT)	Manual usuário – tGo marcha trainer	30
	→ Download das instruções atuais: www.thomashilfen.com	
(IT)	Manuale d'uso – tGo andatura allenatore	38
	→ Download istruzioni aggiornate: www.thomashilfen.com	
(FR)	Manuel de l'utilisateur – tGo démarche formateur	46
	→ Téléchargement des instructions actuelles: www.thomashilfen.com	
(NL)	Gebruikershandleiding – tGo looptrainer	54
	→ Download actuele handleiding: www.thomashilfen.com	
(NO)	Brukerhåndbok – tGo gangart trener	62
	→ Last ned den nyeste bruksanvisningen: www.thomashilfen.com	
(SE)	Bruksanvisning – tGo gång tränare	70
	→ Ladda ner aktuella anvisningar: www.thomashilfen.com	
(DK)	Bruksanvisning – tGo gangart træner	78
	→ Download den aktuelle vejledning: www.thomashilfen.com	
(FI)	Käyttökäsikirja – tGo kävelyn kouluttaja	86
	→ Päivitetyt ohjeet voi ladata osoitteesta: www.thomashilfen.com	
(PL)	Podręcznik dla użytkownika – tGo chodu trenera	94
	→ Pobieranie aktualnej instrukcji: www.thomashilfen.com	
(HR)	Korisnički priručnik – tGo hodalić	102
	→ Preuzimanje aktualne upute: www.thomashilfen.com	

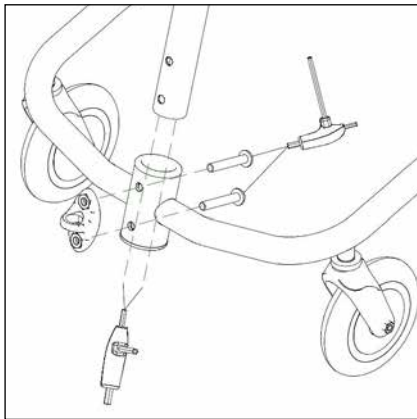
(1)

(2)

(3)

(4)

(5)

(6)

(7)

(8)

(9)

(10)

(11)

(12)

(13)

(14)

(15)

(16)

(17)

(18)

(19)

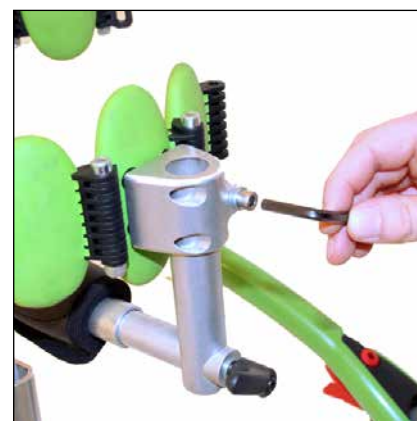
(20)


(21)

(22)

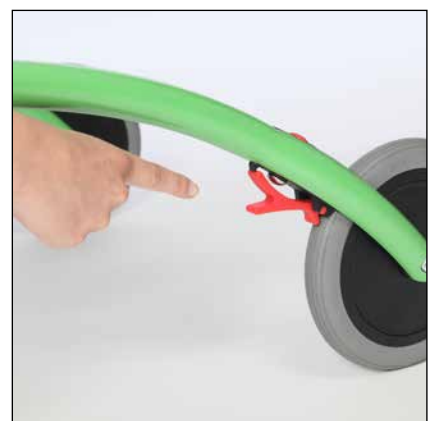(23)

(24)

(25)

(25)

(27)

(28)

(29)

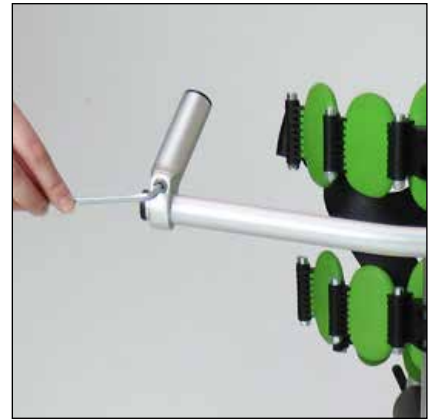
(30)

(31)


(32)

(33)

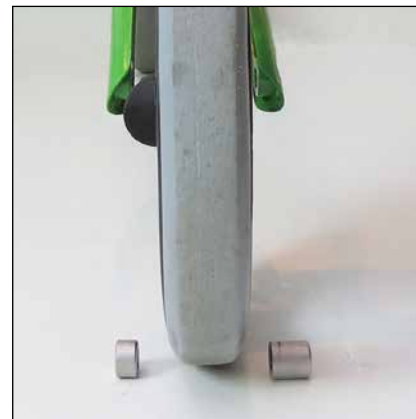
(34)

(35)

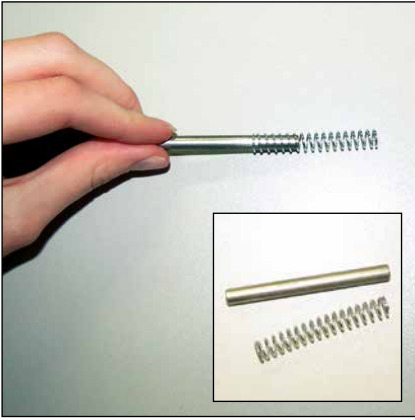(36)

(37)

(38)

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Dear tGo user,

Thank you very much for the trust you have placed in us and for purchasing this product. With your tGo you have acquired an innovative product from the Thomashilfen range.

To ensure the tGo is safe, practical and comfortable for both you and in particular for your child, please read these operating instructions first.

Should you still have questions or problems, please refer to your specialist dealership or contact us directly. In this context we would like to draw your attention to our website at www.thomashilfen.com where you will also be able to access up-to-date information.

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Contents	Page
Permissible operating conditions/places of use	14
Risks of use and contraindications.....	14
Intended purpose / indication.....	15
Safety checks and maintenance intervals.....	15
Notes on safety	15
Symbols and warning signs	16
Technical specifications	16
Basic configuration	17
Accessories.....	17
Setup and assembly	17
Adjusting the gait trainer to the child/range of adjustments	18
Parking brake	19
Friction brake.....	19
Fitting the accessories	19
Product/serial number	21
Notes on cleaning, disinfection and care.....	21
Warranty conditions/CE marking.....	21
Maintenance/reuse	21

Permissible operating conditions/places of use

Before the tGo is used for the first time, it must be instructed by a trained specialist.

CAUTION: The tGo is exclusively meant for secure indoor gait training. Its design is particularly suited for use in living areas.

Risks of use and contraindications

Using this product properly will prevent any risks from arising during use. There are no contraindications associated with proper use.

Intended purpose/indication

Walking/gait training for the following conditions:

- Children with impaired motor development
- Following impairment of the CNS (impairment of the CNS function or following infection/tumors)
- Post traumatic/post infection conditions with disorders of the musculoskeletal system, where applicable with doped relief
- Children with cerebral impairment of the locomotor system (diparesis, tetraparesis)
- Children with spina bifida
- Neurological diseases
- Craniocerebral injuries
- Muscle diseases (congenital/acquired/progressive)
- Metabolic nerve, muscle and bone diseases

Safety checks and maintenance intervals

A regular check of the operating controls and securing screws should be carried out by a trained specialist every month. Please remember to re-secure the screws and locking levers following each adjustment.

Notes on safety

- Before using the tGo for the first time, read the operating instructions carefully – or have them read aloud to you if you have difficulty reading. If you have lost the operating instructions you can download them at any time from the download page of our website at www.thomashilfen.com
- We recommend that you read just the settings of the tGo at least every three months to suit the actual size of your child's body. This should be carried out with assistance from your therapist/orthopedic technician where appropriate.
- Never leave the tGo user unsupervised in the gait trainer!
- The proper function of the parking brake should be checked before each use of the tGo. If the brake does not function properly the tGo may not be used. Please contact your specialist dealer to resolve the problem.
- **CAUTION:** It should be ensured that limbs are not trapped in the gap between the wheel and the wheel fork.
- **CAUTION:** Be aware that there is an increased pinch hazard when making settings and adjustments to the tGo.
- **CAUTION:** Wet wheels can affect the performance of the brakes. During a longer pause you should secure the tGo against unintended rolling by applying the parking brake.
- The gait trainer should only be used on a level, horizontal and firm surface. Be aware of unevenness and holes in the floor. There is a risk of tipping over or becoming stuck.
- You should not hang or place heavy bags or other similar items on the tGo because this will increase the risk of it tipping over.
- Please keep your children away from the packaging – there is a risk of suffocation.
- When accessory parts are removed tube ends may be left open. These may be sharp-edged and therefore present a risk of injuries to fingers that are poked into these openings. The tGo and its accessories have been designed to avoid such risks. However, should open-ended tubes still present a risk they should be closed off using plastic plugs.
- The pad elements must not be eaten.

Symbols and warning signs

Sticker	Meaning
	<p>Specification labels / loading capacity stickers</p> <p>The specification label is attached to the rear of the central tube.</p> <p>The specification label includes the name of the manufacturer, including the address; the type designation, the maximum loading capacity and the CE label.</p>
	<p>Bar code label</p> <p>The barcode label is attached to the front of the base.</p> <p>The bar code label includes the item number, designation, serial number and production date of the product.</p>
	<p>Note on use</p> <p>The instruction for use is attached to the front of the central tube.</p> <p>The sticker refers to the use of the product only indoors and to the reading of the instructions for use.</p>

Technical specifications and versions

	tGo 1	tGo 2
Seat height	250 mm - 600 mm / 9.8 - 23.6"	250 - 800 mm / 9.8 - 31.5"
Overall dimensions	830 x 540 mm / 32.7 x 21.3"	920 x 650 mm / 36.2 x 25.6"
Handgrip height	430 - 900 mm / 16.9 - 35.4"	450 - 1100 mm / 17.7 - 43.3"
Pelvis support height	400 - 750 mm / 15.8 - 29.5"	400 - 950 mm / 15.8 - 37.4"
Wheel size - front	150 mm / 5.9"	150 mm / 5.9"
Wheel size - rear	200 mm / 7.9"	200 mm / 7.9"
Weight	12.4 kg / 27.3 lbs	13.8 kg / 30.4 lbs
Max. user weight	40 kg / 88.2 lbs	45 kg / 99.2 lbs

tGo is available in the following versions:

Item code	Description
1887	tGo gait trainer size 1
1888	tGo gait trainer size 2

Material

Aluminum, stainless steel, steel (painted), plastic, rubber, polyester, polyurethane

Basic configuration

In the basic configuration the tGo is supplied with the following parts:

Subframe with parking brake and friction brake, Height and angle adjustable handgrips, Swiveling saddle, Pads for chest (5 elements), pelvis (5 elements) and trunk (3 elements), Strap for the chest pads, Allen wrench with 2 different end sizes, Castoring front wheels with steering lock

Accessories

The following accessories are available for the tGo:

Seat brake system, Leg separator plate, Additional pads, Split handgrips, Vertical handgrips, Return stop, Pull rod

Setup and assembly

At delivery, the tGo is knocked down into its individual parts. Follow these individual steps to assemble your tGo:

1. Place the wide central tube into the locator on the subframe such that the holes in the tube are aligned with the holes in the locator (fig. 1). Fit the Allen bolts provided through the holes in the frame locator and central tube from the inside (fig. 2).
2. Take the plastic eye with pre-fitted nuts and offer this up to the holes in the front side of the tGo through which the Allen bolts are protruding. Tighten the screws into the nuts in the plastic eye. Ensure that the central tube and subframe are securely fastened together (fig. 3).
3. Tighten the two Allen bolts in the black plastic part inside the lower end of the central tube. Ensure that each bolt is turned fifteen times at least (fig. 4).
4. Now loosen the Allen bolt that is screwed into the front element of the central articulating arm from the side. Press the articulating arm onto the central tube such that the arm is pointing towards the inside of the tGo frame and the bulged part of the front metal element is facing upwards. Align the articulating arm at right angles to the front side of the subframe and fasten with the Allen bolt that is located on the front element (fig. 5). Ensure that the central tube and articulating arm are securely fastened together and that the articulating arm cannot slip downwards under load.
5. Loosen the lower nut that is located on the threaded rod beneath the saddle. Insert the threaded rod into the foremost of the two locating holes in the rearmost element of the articulating arm (fig. 6). In front of the saddle is a tube that points upwards. Ensure that the saddle – together with the tube – is inserted into the locator in the direction of travel. Replace the removed nut onto the threaded rod beneath the articulating arm. In so doing, ensure that the rubber O-ring within the nut is facing upwards (fig. 7). Tighten the nut.
6. Take the handlebars in your hands. Undo the Allen bolt in the joint at the upper end of the handlebar tube. To do this, the locking lever must first be released and the handlebar positioned such that the hole in the handlebar locator aligns with the oblong hole in the handlebar tube (fig. 8). Now the long end of the Allen wrench can be inserted into the tube until it locates into the head of the Allen bolt (fig. 9). Undo the bolt.
7. Insert the handlebar tube into the central tube of the tGo (fig. 10). You can turn the handlebar through 360° within the central tube. This allows you to choose whether the handlebar locating bracket should face forwards (fig. 11) or backwards (fig. 12). Then align the handlebar so that it is parallel with the front side of the tGo. Secure the handlebar by fully tightening the Allen bolt inside the handlebar tube. Place the handlebar in the desired position and lock in-place with the locking lever (fig. 13). Secure the locking lever with the built-in slider.
8. Slot the pelvis pad onto the tube in front of the saddle. To do this, the screw on the tube locator must first be loosened. The Allen bolts that connect the various pads should be fitted such that their heads are pointing upwards. Tighten the screw on the tube locator in order to secure the pads in position (fig. 14).
9. To fit the chest pads, first loosen the screw in the chest pad tube. Slot the tube into the tGo saddle tube such that the chest pads are positioned above the pelvis pads (fig. 15). Secure the tube in-place by tightening the internal screw.
10. To fit the torso pads, first loosen the locking lever on the back of the torso rest. From behind, push the torso rest tube into the tube on which the saddle is mounted. Secure the torso rest with the locking lever (fig. 16).

Adjusting the gait trainer to the child/range of adjustments

Settings/adjustments may only be undertaken by specialists that have received the requisite training or instruction (e.g. a medical product consultant at a medical appliance specialist dealership).

Setting the saddle height

1. First set the height of the articulating arm at the central tube. To do this, undo the screw that is located on the side of the front element and slide the articulating arm to adjust its height (fig. 17). Once the articulating arm has been adjusted, secure it with the screw located on the front element. Ensure that the central tube and articulating arm are securely fastened together and that the articulating arm cannot slip downwards under load.
2. Fine adjustment of the saddle height is achieved via the joints on the articulating arm and the threaded rod beneath the saddle. To adjust the joint angle undo the respective screw that is located in the black area on the upper surface of the joint (fig. 18). Place the articulating arm in the desired position and secure in-place by tightening the screw on the upper surface of the joint.
3. To adjust the saddle height via the threaded rod beneath the saddle, rotate the nuts on the threaded rod above the articulating arm upwards (saddle height is reduced) or downwards (saddle height is increased) (fig. 19). Once you have set the desired height, secure the saddle at that height by rotating the nut below the articulating arm upwards as far as possible and tightening it (fig. 20).

Changing the saddle position

The saddle can be fitted in two different positions. This allows its distance to the handlebar to be changed. To do this, proceed as follows:

1. Loosen the lower nut that is located on the threaded rod beneath the saddle. Insert the threaded rod into the front or rear of the two locating holes in the rearmost element of the articulating arm (fig. 6). Ensure that the saddle – together with the tube – is inserted into the locator in the direction of travel.
2. Replace the removed nut onto the threaded rod beneath the articulating arm. In so doing, ensure that the rubber O-ring within the nut is facing upwards (fig. 7). Tighten the nut.

Adjusting the handlebar

1. First adjust the height of the handlebar by undoing the Allen bolt in the joint at the upper end of the handlebar tube. To do this, the locking lever must first be released and the handlebar positioned such that the hole in the handlebar locator aligns with the oblong hole in the handlebar tube (fig. 8). Now the long end of the Allen wrench can be inserted into the tube until it locates into the head of the Allen bolt (fig. 9).
2. Insert the handlebar tube into the central tube of the tGo until it has reached the desired position. Please note that the handlebar tube may only be pulled out as far as the mark – and no further. You can turn the handlebar through 360° within the central tube. This allows you to choose whether the handlebar locating bracket should face forwards (fig. 11) or backwards (fig. 12). Then align the handlebar so that it is parallel with the front side of the tGo. Secure the handlebar by fully tightening the Allen bolt inside the handlebar tube.
3. Place the handlebar in the desired position (fig. 21) and lock in-place with the locking lever (fig. 13). Secure the locking lever with the built-in slider.

Adjusting the pad height

1. The height of the pelvis pad can be adjusted. To do this, the screw on the tube locator must be loosened. The pelvis pad can now be slid up and down the tube to the desired height. Re-tighten the screw on the tube locator to secure the pelvis pad (fig. 14).
2. The height of the chest pad can be adjusted by moving the tube to which it is fastened. To do this, undo the screw at the top of the chest pad tube (fig. 22) and slide the tube to the desired height. Re-tighten the screw to secure the chest pad.
3. The height of the torso pad can be adjusted. To do this, the screw on the tube locator must be loosened (fig. 23). The torso pads can now be slid up and down the tube to the desired height. Re-tighten the screw on the tube locator to secure the pads.

Function of the torso pad

To make it easier for the child to enter the tGo the torso pad can be rapidly released via the locking lever and pulled out from behind. The locking lever can also be used to adjust the depth of the torso pad. Always ensure that the locking lever is returned to the locked position after making adjustments.

Adjusting the belt

The belt can be adjusted in length using the sliding buckle. (fig. 28) The Velcro fastener at the end of the belt serves to open and close the belt. (fig. 27).

Caution: The Velcro fastener must always be fully closed when the child is in the tGo.

Changing the number of pads

The number of pads can be adjusted flexibly to the child. For instructions see „additional pad elements“ in the section „Fitting accessories“.

Locking/releasing the swiveling function of the saddle

The child must first always be removed from the tGo when changing between the swinging function of the saddle and its fixed position. Then remove the saddle of the tGo. Remove the Allen bolt and washer located at the lower end of the saddle's threaded rod. Now pull the threaded rod out from below. The threaded rod has different shaped notches on each end. When the threaded rod is slotted onto its locator with the rectangular notch at the top, the position of the saddle is fixed and the swiveling function is deactivated.

When the threaded rod is slotted onto its locator with its other at the top, the swinging function is activated. After making this adjustment, refit the saddle to the tGo (for instructions on how to fit the saddle see section 4, "Setup and assembly").

Parking brake

To lock the parking brake, flip down the red levers (fig. 24) located on the underside of the frame in front of the wheels. This locks the brakes (fig. 25). To release the parking brake, flip the red levers up.

Friction brake

The friction brake is applied using the red adjuster on top of the frame in front of the rear wheels. There is a mark on the red adjuster. When this is pointing in the direction of travel the friction brake is released.

The braking action can be set progressively in five stages using the Allen wrench. To do this, turn the red adjuster anti-clockwise (fig. 26). When the adjuster is turned through the fifth position the mark will again be pointing in the direction of travel and the friction brake will be deactivated.

Fitting accessories

Installing and adjusting the vertical handles

Assembly

To install the vertical handles (available as accessories), the standard handlebars have to be replaced. On one side of the handlebars, remove the foam cover by pulling it off the standard handlebars. (Fig. 30) Remove the small plastic screw on the handlebar adapter using a screwdriver (Fig. 31) and set the screw aside. Remove the steering tube by guiding the side without the foam cover through the ring in the handlebar adapter. Loosen the hexagon screw on one of the horns. Remove the horn and slide the handlebars through the ring in the handlebar adapter so that the screw of the handlebar adapter faces away from the tGo (Fig. 32). Retighten the screw on the handlebar adapter. Slide the horn onto the handlebars so that the mounting screw faces away from the tGo and tighten it (Fig. 33). Before using, ensure that the screws are firmly tightened.

Leg separator plate

The leg separator plate is attached to the rearmost element of the articulating arm, where the saddle is fitted. Fit the leg separator plate either to the front or rear of the two locating holes in the rearmost element of the articulating arm. The saddle must be removed before the leg separator plate can be attached. Push the leg separator plate into one of the locating holes in the articulating arm from beneath. Secure the leg separator plate using the nut. Refit the saddle to the tGo (for instructions on how to fit the saddle see section 4, "Setup and assembly").

Additional pad elements

The number of pads fitted to the tGo can be adjusted according to individual requirements. To fit additional pads, the black plastic element – that has a slot in it for the belt – must first be removed. To do this, remove the screw that connects the outer pad with the plastic element (the supplied Allen wrench and a 10 mm ring or open end wrench are required).

Pull the black plastic element out of the pad. Push the desired number of pads onto the pad ring. Ensure that the black plastic pad holders are always pointing away from the child's body. Always secure the pads using the supplied nuts and screws. Finally, replace the belt guide components onto the pad ring and secure in-place.

Follow the same procedure for removing pads that are not required.

Installing and adjusting the seat-brake system

Assembly

To install the seat-brake system, the standard rear wheels have to be replaced with the rear wheels with integrated seat-brake system (available as accessories). This should only be done by a specialist dealer. In order to do so, the hexagon screw that connects the frame of the tGo to each wheel has to be removed (Fig. 36). Now the standard wheels can be replaced by the wheels with integrated seat-brake system. Ensure that the suspension is perpendicular to the floor during installation (Fig. 38). Mount the wheels on the frame of the tGo using the hexagon screws.

If the weight of the child being fitted is very low, a spring may have to be removed from the seat-brake system.

In order to do so, remove the two Allen head screws from the wheel. Set the cap aside, remove one of the metal pins and pull off the spring. (Fig. 37). Set the pin back into the fixture without the spring. Ensure that the spring of the other pin is under tension in the recess. Position the cap with the round side facing down and screw it down tight. Before using, ensure that the screws are firmly tightened.

Installing and adjusting the return stop

Assembly

To install the return stop, the standard rear wheels have to be replaced with the rear wheels with integrated return stop (available as accessories). This should only be done by a specialist dealer. In order to do so, the hexagon screw that connects the frame of the tGo to each wheel has to be removed (Fig. 36). Now the standard wheels can be replaced by the wheels with integrated return stop. Install the wheels so that the plastic curve (Fig. 34) faces to the left respectively. Insert the smaller metal spacer sleeves in this side. Install the wheel on the other side with the larger metal spacer sleeves (Fig. 35). Mount the wheels on the frame of the tGo using the hexagon screws. Before using, ensure that the screws are firmly tightened.

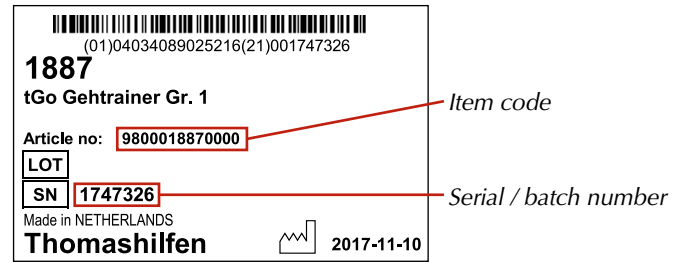
Pull rod

The pull rod can be hooked into the plastic eye on the tGo frame when needed. (fig. 29)

Product/serial number (references the year of manufacture)

To ensure you receive the correct replacement parts or for potential technical product recalls we will always require you to state the serial/batch number together with the exact item number. The serial/batch number will allow you to find out the year of manufacture of your tGo from us.

The serial/batch number and the item number of the tGo can be found on the barcode label on the tGo subframe.

Notes on cleaning, disinfection and care

If necessary you can wipe down all parts of the tGo using a dampened cloth. Household cleaning agents and disinfectants can be used to remove more stubborn dirt.

Warranty conditions/CE marking

From the date of purchase Thomashilfen provides a 2-year warranty on all frame components. The warranty covers all claims that impair the function of the tGo. Damage caused through improper use (e.g. overloading) and normal wear and tear is excluded. The tGo complies with the requirements of international standard DIN EN ISO 11199-3 TZ 4.3 and carries the CE mark. We hope that you enjoy using your tGo gait trainer.

Maintenance/reuse

Servicing and repairs to the gait trainer may only be carried out by a specialist dealership. The user of this rehabilitation aid must ensure that the prescribed inspections/maintenance are/is performed regularly and at the correct intervals. If damage is evident the user is under a duty to actively inform and to have the damage repaired immediately by specialist personnel. Only original spare parts may be used for repairs (the list of replacement/spare parts can be found on the download page of our website at www.thomashilfen.com).

The removal and fitting of spare parts may only be carried out by specialist personnel! If it becomes necessary to return the tGo to your specialist dealership, please ensure that the chassis is properly and securely packed for transportation purposes.

Familiarize yourself with the functions of the product. Should you not already know the product you must study the operating instructions before carrying out any inspections. Clean the product before commencing with inspections. For cleaning, observe the notes on cleaning, disinfection and care.

Subsequently check the tGo for damage/cracks in the frame – including the handlebars, the wheels (including the wheel forks), the saddle, the brakes and all of the pad elements.

The tGo is suitable for reuse. However, this rehabilitation product is subject to exceptional stresses.

We have calculated the product's service life – when used properly and in compliance with the service and maintenance schedule – to be 5 years, on the basis of market observations and the state of the art. Periods during which the product is in storage at specialist dealerships is excluded.

With corresponding care and maintenance this product will also be reliable in operation for significantly longer than this period.

Before reuse the product must always be carefully cleaned and disinfected. The condition of the product must be checked by trained specialist personnel for wear and damage. Any problems with the product must be rectified in order to ensure that the tGo can continue to be used safely. In this context, observe the notes on safety. For reuse, the product must be supplied with its operating instructions. If none are available, either request a copy from Thomashilfen or obtain them from the download page of our website at www.thomashilfen.com

If the condition of the tGo is unsuitable for reuse it can be recycled at your local recycling facility.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG